



iotaSOFT™ Control Console & Accessories

Cleaning and Sterilization Information

The iotaSOFT Drive Unit is supplied for single patient, unilateral use only. DO NOT reuse or sterilize.

To clean the iotaSOFT Control Console & Accessories after use:

1. Disconnect the Foot Pedal, switch off the Control Console, and disconnect from power supply prior to cleaning.
2. Use a soft, lint-free cloth with 70% isopropyl alcohol, CIDEX, or equivalent to clean the exterior of the Control Console, Footpedal, and power supply. NOTE: Abrasive cloths, towels, paper towels, detergents, abrasive cleaners, and similar may damage the equipment.
3. Never immerse device in water or other liquids. Never allow water entry into Control Console, Footpedal, and power supply.
4. Check housing and connecting cables for damage and dirt. Verify the ground pin is present on the power supply cord.



Please consult the IFU for further information and instructions for use.